LABORATOIRES  
**GILBERT**

Fiche technique : Anti-adhésif  
*Data sheet : Anti-adhesive*

LABORATOIRES  
**GILBERT**

**ANTI-ADHESIF**  
**ANTI-ADHESIVE**

**DESCRIPTION / DESCRIPTION :**

---

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*The Anti-adhesive makes easier the adhesive dressing unsticking and enables the adhesive mass traces to be eliminated.*

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*Detach the security seal on the capsule for the first use. Use a small quantity. With a moistened cotton, apply between the skin and the dressing without touching the wound by pulling gently on the sticking plaster. Wipe systematically and carefully any traces of the product after use, in particular on the folds, back, armpits... Make sure that the product did not flow. If this case arises, wipe carefully.*

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*Do not apply on the wounds, injured skins, mucous membranes and around the eyes. Use with precautions on the elderly. Do not use on child under 36 months. For a use before an electrode or electrical bistoury plates putting in, ensure imperatively of the evaporation and elimination of any product traces with a dry swab. Do not apply another product without well cleaning the skin. Do not swallow – External use.*

*Failure to comply with the instructions and precautions for use can involve undesirable effects of burn type.*

### CONDITIONS DE CONSERVATION / STORAGE CONDITIONS :

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*Do not use after the expiry date stated on the packaging. Store at room temperature (25°C), away from light and heat.*

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*If medical advice is needed, have product container or label at hand. Keep out of reach of children. IF SWALLOWED: Immediately call a POISON CENTER/doctor. Do NOT induce vomiting. Store locked up. Dispose of contents/container according to the separated collection system used in your municipality.*

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### **Acute irritant / corrosive effect on the skin (method according to standard EN 10993 - 10)**

*Interpretation of the results suggests that Gilbert Anti-Adhesive is **moderately irritating** to the skin.*

### **Search for skin sensitizing potential (method according to standard EN 10993 - 10)**

*Under the experimental conditions adopted, the "Anti-adhesive" product applied pure can be considered as a **skin sensitizer**.*

*However, the anti-adhesive is not expected to present a major risk of skin intolerance under normal or reasonably foreseeable conditions of use - topical and short-term application to the skin - except in people with specific individual sensitivity.*

*A local and short-lived pro-inflammatory reaction may occur after application of the product. A risk of skin allergy is foreseeable after repeated application of the product.*